



Always perfectly oriented.

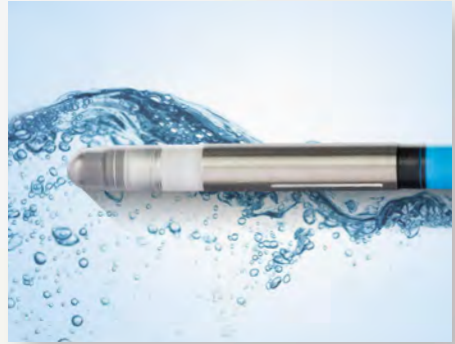

Semiflex 3D

3D Thimble Ionization Chamber
for Relative and Absolute Dosimetry

The New Reference Class
Full 3D Geometry
For FFF and FF Beams

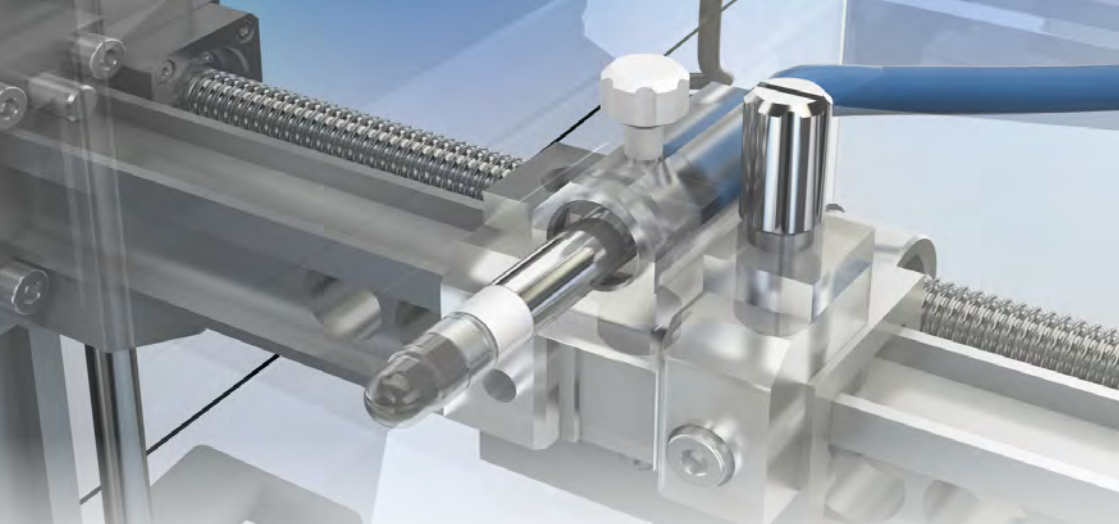

Semiflex 3D

3D Detector Technology

For better results.
For faster scans.

Semiflex 3D combines the advantages of a three-dimensional chamber design with advanced manufacturing technology.

For Multiple Applications

- ▶ LINAC Acceptance Testing
- ▶ TPS Beam Data Commissioning
- ▶ Clinical Reference Dosimetry
- ▶ Point Dose Measurements

Highlights

- ▶ As flexible as its name – waterproof reference-class thimble ionization chamber for relative and absolute dosimetry measurements in water, air or any solid state phantom
- ▶ Accurate measurements of standard and non-standard fields down to a field size of 2.5 cm x 2.5 cm
- ▶ True 3D geometry – fast scanning in cross-plane and in-plane directions without tank rotation
- ▶ Vertical chamber holder with convenient stop thimble for quick detector setup and accurate, high-resolution scanning in axial orientation
- ▶ Small size, minimal volume averaging, high spatial resolution – ideal for absolute dose measurement in small and FFF fields
- ▶ Exceptional dosimetric characteristics – meets or exceeds all requirements of AAPM TG-51 and IEC 60731
- ▶ Absolute dose calibration according to all major dosimetry protocols, such as IAEA TRS-398, AAPM TG-51, DIN 6800-2

Relative Dosimetry and Point Dose Measurements



Flexible Orientation

Semiflex 3D has been designed for axial and radial beam incidence, offering greater flexibility and helping to reduce positioning errors that might occur by repeated tank and detector setups. A vertical chamber holder with a convenient stop thimble makes it very easy to quickly mount Semiflex 3D



Perfect for axial and radial profile measurements

perpendicular to the radiation beam. Minimizing stem and cable effects, it allows high-resolution profile measurements in axial orientation.

Detector setup can be easily changed from axial to radial orientation within a few seconds.

Full 3D Geometry


Uniform response in any direction

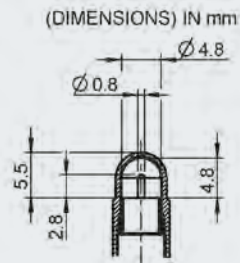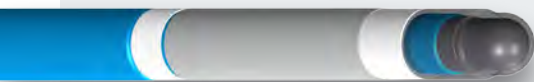

With its true 3D geometry, which provides a high resolution independent of detector orientation, Semiflex 3D allows you to measure in-plane and cross-plane beam profiles much faster and more conveniently.

Using Semiflex 3D, you no longer need to rotate the water tank or change the detector setup to achieve a uniform response, helping you to save setup time and eliminate potential positioning errors.

Clinical Reference Dosimetry in Small and FFF Fields

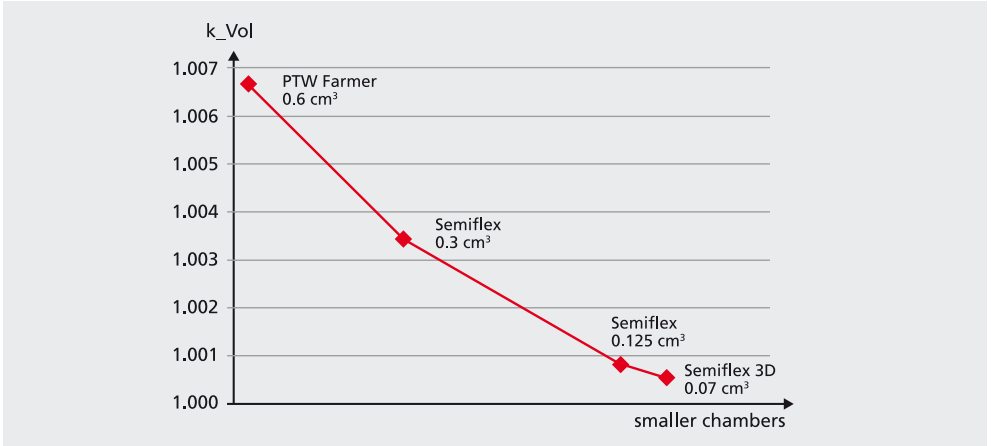

Minimal Volume Averaging: Better than a PTW Farmer®

Illustration comparing estimated volume averaging correction factors (k_{Vol}) for different PTW ionization chambers calculated for 10 MV FFF at D_{max} depth using the geometrical method of [Kawachi2008].

[Kawachi2008]: T. Kawachi et al., Reference dosimetry condition and beam quality correction factor for CyberKnife beam, Med. Phys. **35** (2008), 4591

Compared to a 0.6 cm^3 PTW Farmer® chamber typically used for absolute dose calibration, the small-size 0.07 cm^3 Semiflex 3D has reduced volume averaging to a minimum, making it particularly suitable for high-resolution dose measurements in unflattened (FFF) beams.

The New Reference Class

Exceptional dosimetric characteristics – outstanding accuracy

Well engineered and reliably produced, Semiflex 3D has been designed to meet and exceed all requirements of a reference-class ionization chamber as defined in AAPM TG-51 and IEC 60731.

- ▶ Minimal polarity effect (< 0.4 % correction, < 0.5 % maximum variation)
- ▶ Extremely low volume effect (est. \approx 0.05 % correction in FFF field)
- ▶ Excellent settling time (< 0.3 %)
- ▶ Very low post-irradiation leakage – far below IEC 60731 requirements
- ▶ Significantly reduced cable effects (< 100 fC / (Gy * cm))
- ▶ Minimal chamber-to-chamber variation – reliable k_Q and k_S values

Specifications

Semiflex 3D Type 31021

Detector type:	Vented cylindrical ionization chamber
Design:	waterproof, fully guarded
Measuring quantities:	absorbed dose to water, air kerma, exposure
Reference radiation quality:	^{60}Co
Nominal sensitive volume:	0.07 cm ³
Reference point:	on chamber axis, 3.45 mm from chamber tip
Direction of incidence:	axial, radial
Nominal response:	2 nC/Gy
Long-term stability:	≤ 0.3 % over 2 years
Chamber voltage:	± (50 ... 400) V useful range, ± 500 V max.
Polarity effect:	Photons ≤ ± 0.8 % Electrons ≤ ± 1 %
Directional response:	≤ ± 0.5 % for rotation around chamber axis ≤ ± 1 % for tilting of axis up to ± 70°
Leakage current:	≤ ± 4 fA
Cable leakage:	≤ 100 fC/(Gy·cm)
Wall of sensitive volume:	0.57 mm PMMA, 1.19 g/cm ³ 0.09 mm graphite, 1.85 g/cm ³
Total wall area density:	84 mg/cm ²
Dimension of sensitive volume:	radius 2.4 mm, length 4.8 mm
Central electrode:	Al 99.98, diameter 0.8 mm
Build-up cap:	PMMA, thickness 3 mm

Ion collection efficiency at nominal voltage

Ion collection time:	118 μs
Max. dose rate:	≥ 99.5 % saturation 6.7 Gy/s ≥ 99.0 % saturation 13.4 Gy/s
Max. dose per pulse:	≥ 99.5 % saturation 0.68 mGy ≥ 99.0 % saturation 1.42 mGy
Radiation quality:	^{60}Co ... 50 MV photons (9 ... 45) MeV electrons
Field size:	(2.5 x 2.5) cm ² ... (40 x 40) cm ² (3.0 x 3.0) cm ² ... (40 x 40) cm ² ≥ 18 MV
Temperature:	(10 ... 40) °C, (50 ... 104) °F
Humidity:	(10 ... 80) %, max. 20 g/m ³
Air pressure:	(700 ... 1060) hPa
Connecting system:	BNT, TNC, M

Options & Accessories

T48012	^{90}Sr Radioactive Check Device
T48002.1.004	Semiflex 3D Chamber Holder for Check Device
T41023.1.280	Adapter for PTW Water Phantom 41023 for horizontal beams
T2967/U6	Acrylic PMMA Chamber Plate 30 cm x 30 cm
T29672/U33	RW3 Chamber Plate 30 cm x 30 cm
TRUFIX and TRUFIX ⁸⁵ Holders, Build-up caps, Chamber Plates for OCTAVIUS [®] 4D and more upon request	

Dosimetry Pioneers since 1922.

It all started with a brilliant invention – the revolutionary Hammer dosimeter in 1922. Ingenuity coupled with German engineering know-how shaped the company's history and led to innovative dosimetry solutions that later became an industry standard. Over the years, PTW has maintained its pioneering spirit, growing into a global market leader of dosimetry applications well known for its outstanding quality and precision. Today, PTW dosimetry is one of the first choices for healthcare professionals in radiation therapy, diagnostic radiology, nuclear medicine and health physics.

For more information on PTW dosimetry products, visit www.ptw.de or contact your local PTW representative:

Headquarters

PTW-Freiburg
Physikalisch-Technische
Werkstätten
Dr. Pychlau GmbH
 Tel. +49 761 49055-0
info@ptw.de
www.ptw.de

PTW Dosimetría Iberia S. L.
 Tel. +34 96 346 2854
info@ptwdi.es
www.ptwdi.es

PTW-Asia Pacific Ltd.
 Tel. +852 2369 9234
info@ptw-asiapacific.com
www.ptw-asiapacific.com

PTW-UK Ltd.
 Tel. +44 1476 577503
sales@ptw-uk.com
www.ptw-uk.com

PTW-New York Corporation
 Tel. (1-516) 827 3181
ptw@ptwny.com
www.ptwny.com

PTW-Beijing Ltd
 Tel. +86 10 6443 0746
info@ptw-beijing.com
www.ptw-beijing.com

PTW-France SARL
 Tel. +33 1 64 49 98 58
info@ptw-france.com
www.ptw-france.com

PTW-Latin America
 Tel. +55 21 2178 2188
info@ptw.com.br
www.ptw.com.br

PTW Dosimetry India Pvt. Ltd.
 Tel. +91 44 42079999
info@ptw-india.in
www.ptw-india.in